

## ***Installation Instructions for ONYX Reagent Set***

*Required ONLY when changing to CDS reagents from other non-CDS products.*

---

To achieve optimal performance with CDS *NextGeneration*<sup>™</sup> Reagents, it is important to completely prime the Coulter<sup>®</sup> ONYX instrument. To do so, simply follow the steps below:

1. Install the CDS ONYX/C Reagent Set (CDS 501-026)
2. Access the Setup screen and select F3, PRIME REAGENTS
3. Use the space bar to select Y for both PACK and CLEANER
4. Press F2 to proceed with the PRIME CYCLE
5. Allow the instrument to PRIME
6. After the instrument has finished the first PRIME CYCLE, repeat steps 2 through 5 a second time.

**The instrument is now completely primed with  
CDS *NextGeneration*<sup>™</sup> Reagents.  
Proceed with normal instrument operation, complete reagent log.**

If you experience any problems or have any questions, please call 800-453-3328.