

Installation Instructions for T Reagent Set

Required ONLY when changing to CDS reagents from other non-CDS products.

To achieve optimal performance with CDS *NextGeneration*[™] Reagents, it is important to completely prime the Coulter[®] T Series instrument. To do so, simply follow the steps below:

For the JT, JT II and JT III:

1. Install the CDS T/C Reagent Set (CDS 501-025)
2. Perform two (2) F-16's to prime the Diluent
3. Perform two (2) F-17's to prime the Lyse
4. Perform two (2) F-18's to prime the Cleaner
5. Perform the STARTUP procedure

For the T-540, T-660 and T-890:

1. Install the CDS T/C Reagent Set (CDS 501-025)
2. Perform two (2) F-14's
3. Perform STARTUP procedure

The instrument is now completely primed with

CDS *NextGeneration*[™] Reagents.

Proceed with normal instrument operation, complete reagent log.

If you experience any problems or have any questions, please call 800-453-3328.