

## **Installation Instructions for Ac.T DIFF REAGENT PACK**

Required ONLY when changing to CDS reagents from other non-CDS products.

1. Access the DIAGNOSTIC FUNCTIONS screen.
2. Press the PREPARE TO SHIP icon.
3. Follow the Operations Manual to execute steps 1 - 4.  
Note: It is not necessary to place the cleaner pick-up tube in and disinfectant prior to step 2.
4. After completing step 4 power down the instrument.
5. Connect the reagent lines to the CDS reagents.
6. Install the Reagent Card supplied with the Ac.T DIFF REAGENT PACK.
7. Power up the instrument
  - a. The diluent will self prime.
  - b. The lyse will require priming. Press the L icon when prompted.
  - c. The cleaner will require priming. The instrument will provide the C prompt when the first shutdown is requested.
8. Perform 2 start-up cycles. Insure the background counts are in specification.

**The instrument is now completely primed with  
CDS *NextGeneration*<sup>™</sup> Reagents.  
Proceed with normal instrument operation. Complete reagent log.**

If you experience any problems or have any questions, please call 800-453-3328.